Optelec Compact +

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on 13th February 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will allow us to help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

RoHS Compliance

This product is in compliance with Directive 2002/95/EC of the European Parliament and of the Council of 27 January 2003, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and its amendments.

European Union Notice

Products with CE marking comply with the R&TTE Directive (99/5/EC), the EMC Directive (89/336/EEC), and the Low Voltage Directive (73/23/EEC) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following international standards:

IEC 60950-1 2004;
IEC 55022
IEC 55024 (IEC61000-4-2 excluded)

FCC
FCC part 15 Class B

Compact⁺

User Manual

OPN: COMP-WRLD Version 2.4.2
© 2007 Optelec, the Netherlands
All rights reserved

Optelec
P.O. Box 399
2990 AJ Barendrecht
The Netherlands
Phone: +31 (0)88 678 34 44
E-mail: info@optelec.nl
Internet: www.optelec.com

Contents

1. Introduction	3
About this manual.....	3
What's in the box?.....	4
2. Getting to know your Compact⁺	5
Front side	5
Bottom side	6
Right side	7
Underside.....	8
3. Getting started	10
Placing the batteries.....	10
Turning power on / off	11
Using the magnification dial	12
Selecting a viewing mode.....	12
Using the snapshot button.....	13
Writing with the Compact ⁺	14
Using the collapsible, swing-out handle	14
4. Power and battery use	15
Charging the batteries	15
System Auto Off	16
Appendix A: Safety.....	17
Appendix B: Technical information	18

1. Introduction

Congratulations on choosing the Compact⁺, a pocket size electronic video magnifier designed by Optelec.

Optelec's pocket size electronic magnifier enlarges, with magnification up to 10 times, and enhances printed text on a 4.3" widescreen display.

With its one button simplicity, controlling both magnification and viewing modes, the Compact⁺ offers an enhanced design that extends in operating comfort. It is designed for everyday use at any location.

If you have any questions or suggestions concerning the use of this product, please contact your distributor or the Optelec headquarters using the contact information at the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy using your Compact⁺.

About this manual

This manual will familiarize you with the features and basic operations of the Compact⁺. Please read this manual carefully before using your device.

What's in the box?

The Compact⁺ packaging contains the following:

- The Compact⁺ pocket size electronic magnifier
- A detachable neck cord
- A protective carrying pouch
- A power cable with power adapter and 4 plugs (EU, UK, US and AUS)
- This user manual
- The Compact⁺ rechargeable AA battery pack


If any of these items are missing from your package, please contact your distributor.

2. Getting to know your Compact⁺

When the unit is placed correctly in front of you the display will be facing up and the round magnification dial / mode button is at the right side of the unit.

Front side

The following image shows the front side of the Compact⁺.

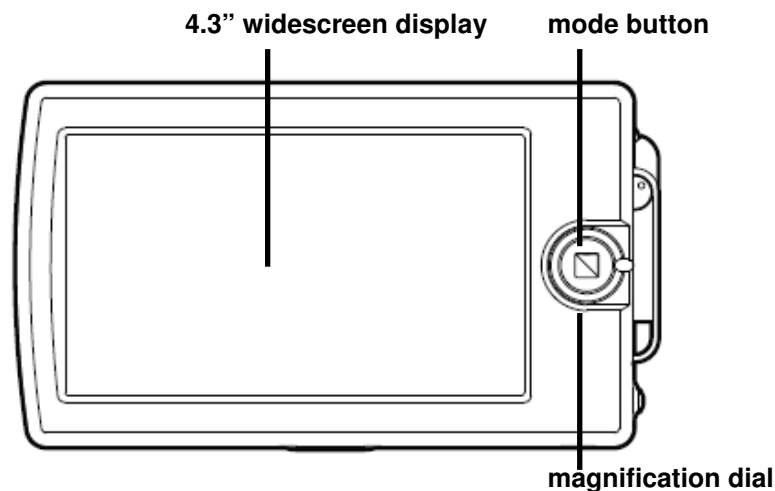4.3" widescreen display

The 4.3" widescreen displays text and images

Mode button


Pressing the mode button allows you to select one of the five viewing modes

Magnification dial

Turning the lever allows you to adjust the magnification

Bottom side

The following image shows the bottom edge of the Compact⁺.

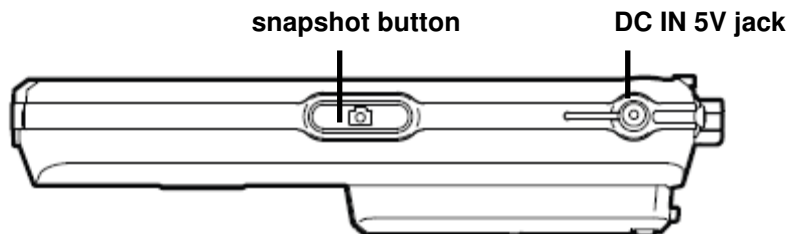**Snapshot button**


The snapshot button allows you to temporarily freeze an image on the screen of an object or text that may be at arm's length, enabling you to bring the image closer to your eyes

DC IN 5V jack

The AC power adapter cable connects to this jack. Use only the model of AC adapter that comes with the Compact⁺. Use of an adapter other than that supplied with the unit automatically voids the warranty.

Right side


The following image shows the right side of the Compact⁺.

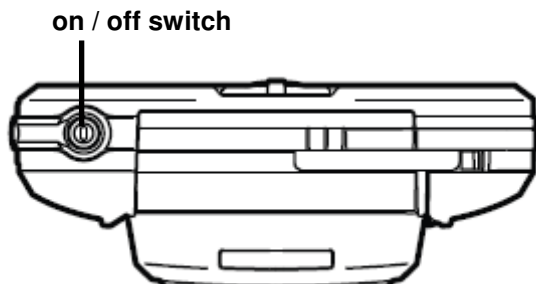**On / Off switch**

Pressing the small round orange button for half a second will switch the unit on or off

Underside

The following image shows the underside of the Compact⁺.

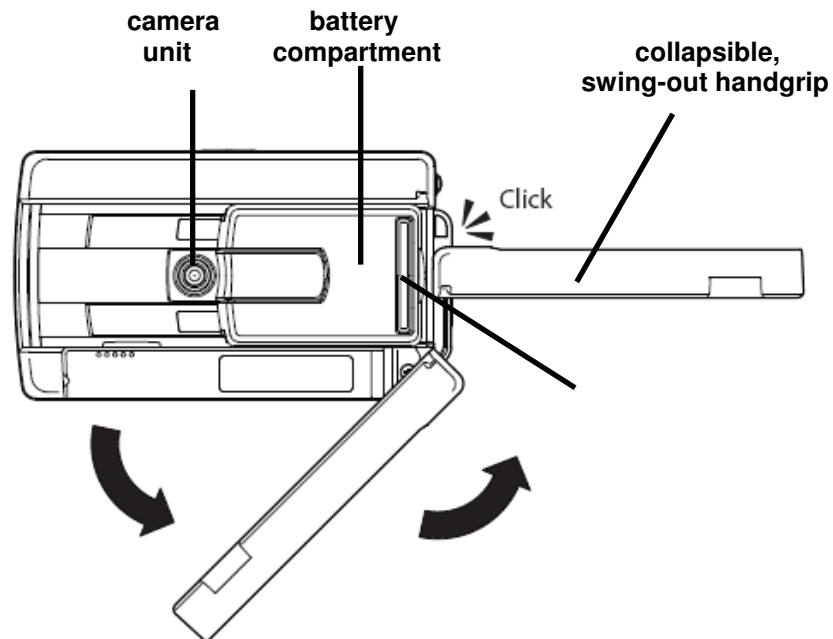Battery compartment

The battery compartment can take the Compact⁺ battery pack supplied with the unit or two single AA batteries, either rechargeable or disposable

Camera unit

The Compact⁺ features an auto-focus camera that can slide in two positions; one for reading ('centre') and one for writing ('end')

Collapsible, swing-out handgrip

Pull and swing-out the collapsible handgrip for enhanced reading comfort

3. Getting started

Placing the batteries

Your Compact⁺ will operate on one of three battery options:


1. The Compact⁺ battery pack supplied with the unit, or
2. AA rechargeable batteries, or
3. AA disposable batteries

1. The Compact⁺ battery pack supplied with the unit consists of two AA rechargeable batteries.

We strongly advise you not to remove, open or disassemble the plastic sealing of the battery pack. Removing this sealing might cause malfunctioning of the batteries.

In the event that the unit has not been supplied with battery pack installed, install the battery pack as follows:

- Turn the unit upside down
- Press the small tab at the far end of the battery compartment to release the cover
- Lift the cover
- Install the battery pack
- Replace the cover of the battery compartment
- Gently press down to lock, until you hear a click

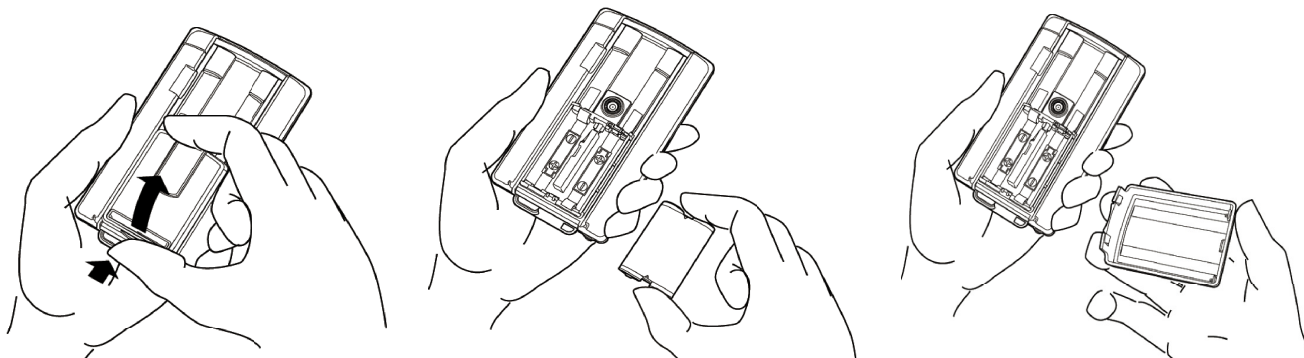For optimal battery life please do not charge the Compact⁺ before the “battery empty” () warning is displayed on the screen. Frequent charging of a full battery pack may damage the battery and reduce its lifetime.

2. and 3. When using AA rechargeable or disposable batteries:

Install the batteries as follows:

- Open the battery compartment as above
- Remove the Compact⁺ battery pack
- Install your batteries being careful to match polarity (i.e. + with +)
- Replace the cover

Please note that when utilising rechargeable batteries, other than those supplied with the Compact⁺, it is not possible to recharge them using the AC power adapter supplied with the unit. Doing so may result in damage to the unit and automatically voids the warranty.

Turning power on / off

Switch the unit on and off by pressing the orange button, located at the right side of the unit, for half a second.

Using the magnification dial

To set the magnification, simply turn the lever of the dial, located next to the display. Placing the unit on a flat surface, allows you to select one of the three magnification levels; 5X, 7.5X or 10X. Lifting the unit allows you to lower the magnification which is useful to obtain an overview of the object or text being viewed.

Selecting a viewing mode

Pressing the round mode button, located in the center of the magnification dial, allows you to select one of the five viewing modes.

- Negative mode: reading in high contrast white on black
- Positive mode: reading in high contrast black on white
- Photo mode: view full color text and photos
- High contrast yellow text on a black background: reading in high contrast yellow on black
- High contrast yellow text on a blue background: reading in high contrast yellow on blue

Pressing the round mode button for approximately one second, allows you to switch object lighting off. Object lighting can be turned on and off in each of the above mentioned viewing modes, facilitating reading of glossy magazines, displays and screens. Pressing the mode button for one second again, will switch object lighting back on, whereas selecting the next viewing mode by a short press of the mode button will turn the lighting on too.

Using the snapshot button

The snapshot button allows you to temporarily freeze an image on the screen.

- Hold or place the Compact⁺ over the text or image
- Press the oval shaped snapshot button, located at the bottom side of the unit
- Bring the unit closer to your eyes to view the object or text
- Turn the lever, located next to the display, to change the magnification
- Press the viewing mode button, in the centre of the magnification dial, to change the viewing mode
- Press the snapshot button again to unfreeze the image

Writing with the Compact⁺

You are also able to use the Compact⁺ for writing. Take the unit in your hand and turn it upside down (display facing down)


- Slide the battery compartment to the centre bringing the camera lens to the far end (see picture).
- Place the unit back on your paper (display facing up).
- Hold the Compact⁺ down on the side of the magnification dial / mode button so that the Compact⁺ rests on that side edge. This lifts the camera and allows a pen to be used within the area shown on screen.
- Place your pen under the raised side and you can start writing.

Slide the battery compartment to its original position to return to the reading mode.

Using the collapsible, swing-out handgrip

The Compact⁺ has a collapsible, swing-out handgrip;, giving you the feeling you are using a regular magnifying glass. Simply pull the handle, located on the underside to release it, and swing it out to eventually snap in place. The collapsible handle gives you additional support and control for optimal reading comfort.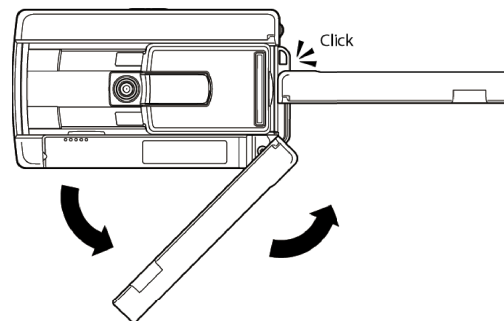


4. Power and battery use

When the AC power adapter is disconnected, the Compact⁺ power source is a battery pack of two rechargeable AA batteries. Off-the-shelf AA batteries can also be used, either rechargeable or disposable. In the interests for the environment and optimal power, we advise you to use rechargeable batteries.

Charging the batteries

To charge the Compact⁺ battery pack supplied with the unit, connect the adapter supplied to the DC IN 5V jack, located on the bottom side and then plug it into a mains power outlet. The orange LED indicator next to the adapter socket will glow continuously while the battery pack is charging. When the battery pack is fully charged the orange LED light will switch off. If the LED indicator starts to flash, it means there is a charging problem. Disconnect the adapter, check if the battery pack is placed correctly in the compartment and try again.

Charging the Compact⁺ takes approximately five hours. A fully charged Compact⁺ will operate for approximately two hours and thirty minutes. Please note that, for safety reasons, only the batteries supplied with the Compact⁺ can be charged in the unit's battery compartment.

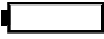For optimal battery life please do not charge the Compact⁺ before the “battery empty” () warning is displayed on the screen. Frequent charging of a full battery pack may damage the battery and reduce its lifetime.

System Auto Off

The Compact⁺ will automatically turn off if it is not used for a period of 4 minutes. Pressing the orange button at the right side will switch the unit on again.

Appendix A: Safety

- Do not expose the Compact⁺ to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the Compact⁺. Contact an Optelec authorized service organisation in your area.
- To avoid risk of electrical damage, keep your Compact⁺ away from fluids and chemicals.
- Handle the Compact⁺ with care. Rough handling will damage the internal components.
- Do not use the Compact⁺ near inadequately shielded medical devices.
- Contact your Optelec partner to service this equipment if necessary. Do not open the Compact⁺ as it will void the warranty.
- Always unplug the Compact⁺ before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.
- Use only the supplied power adapter.
- Do not remove the battery pack when the device is charging.

Use of the Compact⁺ other than described in this manual will exclude it from warranty conditions.

Appendix B: Technical information

Magnification	5X, 7.5X and 10X (placed on a flat surface)
Viewing modes	High contrast white on black High contrast black on white Full color photo mode High contrast yellow on blue High contrast yellow on black
Focus	Auto-focus
Screen	4.3" widescreen TFT
Screen resolution	480 x 272 pixels
Battery	up to 2.5 hours continuous use Charging time 5 hours
Ni-MH battery	Optelec, type EP-1, DC 2.4V
Weight	300 grams / 10.6 ounces (excl. adapter)
Power supply type	Group West, TRC-05-2000-M
Input	100 – 240V, 50 – 60Hz, 0.3A
Output	DC 5V, 2A

Operation conditions:

Relative humidity: <70 %, no condensation

Temperature: 0 to 40 °C

Storage and transport conditions:

Relative humidity: <95 %, no condensation

Temperature: -10 °C to 40 °C

Note: Used batteries should be disposed at a depot that is designated by local competent authorities.

WARNING: The use of accessories and cables other than those specified and sold by the manufacturer of the Compact⁺ as replacement parts for components, may result in increased emissions or decreased functioning of the Compact⁺.

This product complies with the safety and EMC requirements of the directives:

IEC 60950-1 2004;

IEC 55022

IEC 55024 (IEC61000-4-2 excluded)

FCC

FCC part 15 Class B

This device bears the CE and FCC approval mark

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Optelec Limited
Optelec GmbH