

## Setting up the ClearView C Flex



1



2



3



4



5



6



7



8

1. Mount the bottom part of the ClearView C Flex on a stable table and fasten it with the screw;
2. Place the top part of the ClearView C Flex on the bottom part;
3. Use the included Allen keys to tighten the screws at the top and bottom part of the arm. In case the Clearview C Flex is used with the reading table, mount the arm at the top of the pole, in its highest position;
4. Mount the display adjustment and fasten the screws;
5. Place the monitor on the display adjustment;
6. Slide the camera box into its position at the end of the arm. If the camera does not look straight down onto the table surface, adjust the viewing angel using the adjustment screw on top of the camera box holder;
7. Insert the cables;
8. Switch on your ClearView C Flex and it is ready to use.